
OTOSAN® NASAL SPRAY

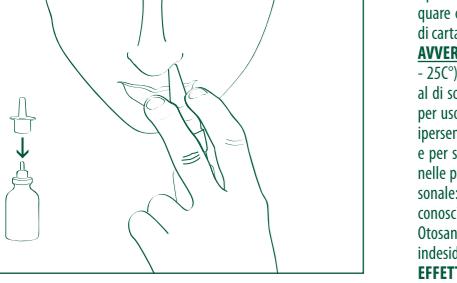IT

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**A 2,2% HYPERTONIC SEA WATER
SOLUTION CONTAINING SEA WATER
FROM BRITTANY, ALOE VERA
AND BLACKCURRANT PLANT EXTRACTS
AND MOUNTAIN PINE, MELALEUCA
(TEA TREE) AND LEMON ESSENTIAL OILS.**

USE. The Otosan® Nasal Spray medical device is a decongestant for topical use. It acts locally by deeply cleansing the nasal cavities, thus helping to reduce congestion, inflammation and the incidence of infection. Otosan® Nasal Spray is a hypertonic (2.2%) sea water solution enriched with plant extracts and essential oils from organically grown plants. Free from propellant gas.

INDICATIONS. Otosan® Nasal Spray is ideal as the first adjuvant therapy for sinusitis and inflammation, such as different types of colds, rhinitis, and for post-surgery.

DOSAGE. 1-2 sprays per nostril up to 4 times a day.

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A MULTI-ACTION PRODUCT TO DEEPLY CLEANSE NASAL CAVITIES WITH A DECONGESTANT EFFECT.

SEA WATER (CONCENTRATE)
= CLEANSING AND SOFTENING ACTION

The sea water contained in this formula comes from the sea near Brittany. Its composition is similar to our blood plasma and it is particularly rich in mineral nutrients and trace elements that are essential for our body. In addition to cleaning the nasal cavities and removing dust and pollen, hypertonic sea water favours the detachment of mucus.

GLYCERIC BLACKCURRANT EXTRACT
= SOOTHING ACTION

Blackcurrant is renowned for its soothing action and for its cortison-like effect.

ALOE VERA DRY EXTRACT
= PROTECTIVE AND HYDRATING ACTION

The polysaccharides in the mucous membrane form a layer that protects against irritants and microorganisms.

MOUNTAIN PINE ESSENTIAL OIL
= BALSAMIC ACTION

Inhaling Mountain Pine essential oil liquefies phlegm and helps to remove it. It is therefore a recommended adjuvant for treating colds, sinusitis, flu, coughs, laryngitis, acute and chronic bronchitis, flu symptoms and asthma.

DESCRIPTION. When brought into contact with blocked nasal mucosa, the hypertonic solution causes watery discharge, draws excess discharge by osmosis, dilutes accumulated viscous mucus by liquefying it and contributes to normalising the oedematous cells. The reduction of mucus viscosity improves mucociliary clearance and helps to remove viruses, bacteria, pollen, as well as macro and micro particles. By eliminating abnormal discharge and treating the mucosa with natural substances that create a hostile environment for bacterial (or microbial) growth, Otosan® Nasal Spray provides relief and prevents relapse.

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WARNINGS AND PRECAUTIONS. Use at room temperature (20° -

25°C). Do not expose the bottle to heat. Do not use on children under the age of 12. Keep out of children's reach. For external use only. Avoid contact with eyes. Do not use in case of known hypersensitivity to any of the components. Do not swallow. Do not use for any other purpose and in different ways than those specified by the manufacturer in these directions. The bottle is strictly for personal use: it might carry infections. Currently, there are no known contraindications of Otosan® Nasal Spray components when used during pregnancy or breastfeeding. SIDE EFFECTS. To date, there are no known side effects.

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